

FOR MAXIMUM PROTECTION AND COMFORT, INSERT HANDFORMED (FOAM) EARPLUGS AS FOLLOWS:

1. Ensure that your hands and earplugs are clean before using.
2. Roll, rather than squeeze, each plug into as small a cylinder as possible.
3. Access your ear canal by reaching over your head with the opposite hand and pulling your ear up and out.
4. Insert the earplug into the ear, gently insert the orange end first when using the orange and green Sound Guard™ foam earplug. Hold gently in place with your fingertip until expansion is complete.
5. When you have a good fit and seal, only the green color will be visible and the orange portion of the earplug will be completely in the ear canal.
6. Do not use handformed earplugs where hazardous chemical vapors can be absorbed into the plug.
7. Dispose of these earplugs after one or two uses.



Size and Ordering Information:

One-size (medium) Sound Guard, two color orange/green NSN 6515-00-137-6345

Amigo (small) yellow pillow pack, 200 pair Contractor Part # CS7981730-WYSTVH, GSA Contract # GS-06F-0070R (ends 7 AUG 2015)

Grande (large) yellow pillow pack, 200 pair Contractor Part # CS7981729-WYST19, GSA Contract # GS-06F-0070R (ends 7 AUG 2015)

