


FOR MAXIMUM PROTECTION AND COMFORT, INSERT TRIPLE-FLANGE EARPLUGS AS FOLLOWS:

1. Using the seating device (lid of the earplug carrying case), place the stem of the triple-flange earplug into the device's open end.
2. Gently push the plug into the ear canal until you get a good seal. The plug is properly fitted when the first two flanges are in the ear canal and the outside third flange is blocking the canal entrance.
3. Check for proper fit by gently tugging on each earplug. You will feel some tension.
4. If not using the seating device, access your ear by reaching over your head with the opposite hand and pulling the ear up and out.
5. Grasp the plug by its stem and insert into the ear canal. Push and twist the plug until you get a good seal.
6. If you cannot get a good seal, try a different size of triple-flange earplug or other type of earplug.

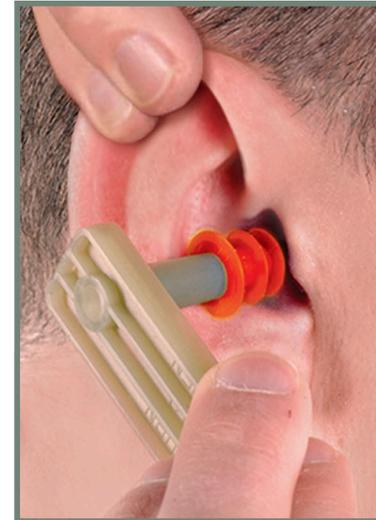

Suggested Size and Ordering information:

10% = Small (green) NSN 6515-00-442-4821
70% = Medium (orange) NSN 6515-00-442-4818
20% = Large (blue) NSN 6515-00-467-0092
Earplug case with inserter NSN 6515-01-100-1674

